

Reprocessing Ultrasound Probes

AUTOMATION

1



2



3



4



5



Reprocessing Ultrasound Probes

AUTOMATION

Always don appropriate PPE

- Reference potential hazards
Reference IFU (Instructions For Use) for cleaning/HLD
- Complete point-of-care cleaning at patient beside before transport to reprocessing area per manufacturer's IFU

1 Transport to reprocessing suite in a properly labeled biohazard device

2 Remove procedure cover

3 Insert soiled ultrasound probe in Ethos® — follow on-screen prompts

4 Remove ultrasound probe and dry with approved drying method to prevent recontamination of ultrasound probe

5 Store in HEPA-filtered cabinet or transport ultrasound probe to next procedure